

MedGyn Balloon Catheter

MEDGYN PRODUCTS, INC.

100 W. Industrial Rd. Addison, IL 60101 USA
t: +1 630.627.4105 toll-free: 800.451.9667
f: +1 630.627.0127 e: info@medgyn.com
w: medgyn.com



The MedGyn Postpartum Balloon Catheter is a disposable, multiple lumen catheter attached to an inflatable balloon system. It is designed to provide tamponade to control hemorrhaging from the uterus and vagina.



Dual Lumen Catheter
and Pear Shaped Balloon



Syringes



Bag Spike



Three Way Stopcock


Features & Benefits

1. 100% silicone for patient comfort
2. Pear-shaped balloon seals internal cervical os
3. Dual lumen catheter aids in monitoring hemorrhage
4. Two syringes for rapid inflation of the balloon
5. Bag spike with check valve
6. Three-way stopcock for, safe and rapid inflation
7. Balloon can intake up to 5000mL of saline (never gas)
8. Includes:
 - 24 Fr balloon tamponade
 - 59cm catheter with three-way stopcock
 - Bag spike
 - Two 50mL luer-locked syringes

Balloon Inflation and Placement

Option One: Direct Saline Inflation

1. Spike a 500mL bag of saline
2. Prime the tubing by attaching the syringe and drawing fluid
3. Use the three-way stopcock to connect the syringe to the balloon catheter
4. Push fluid into the balloon

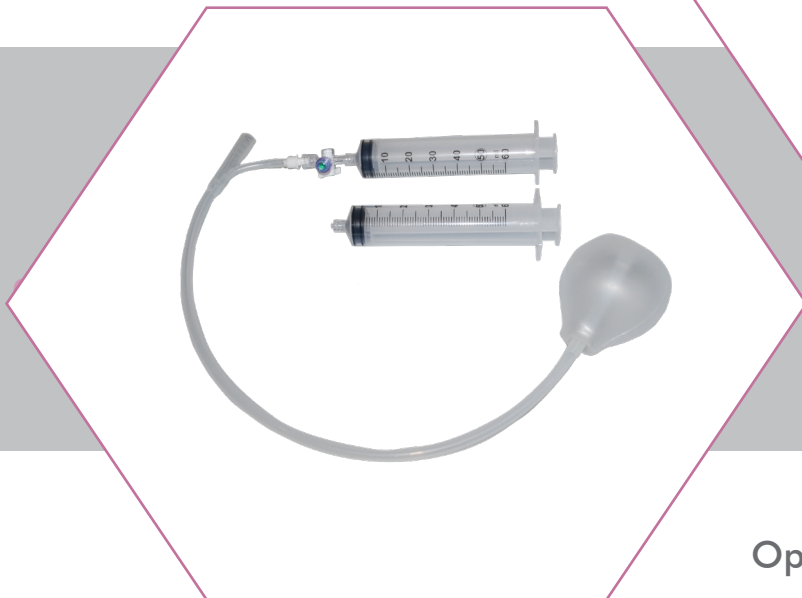

Option Two: Indirect Saline Inflation

1. Draw saline fluid into the syringe
2. Connect the syringe to the balloon catheter
3. Push fluid into the balloon
4. Use both 60mL syringes to inflate the balloon

022703 MedGyn Postpartum Balloon

TRUSTED PRODUCTS FOR PREMIER WOMEN'S HEALTHCARE